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Product: **REPLISIL 17 N**  
Dubliersilikon / duplicating silicone

## Instruction for use

Please read carefully and retain for future reference

Working  
room temperature  
at 23° C

Delivery form:  
2x1 Kg bottles  
2x5 Kg cans

2x6 Kg cans  
2x25 Kg cans

## Indications for use

**REPLISIL 17 N** is an addition-curing 1:1 duplicating silicone with final hardness of Shore A 17, low viscous with best mechanical characteristics and offer an extremely high accuracy in dimension and design of the duplicating form.

## Working instructions

REPLISIL component A and B must be mixed at room temperature in proportion 1:1 approx. 1 min. until an uniform colour is reached. Pour it from a height of approx. 30 cm constantly onto the dry plaster model in the flask until it's all round covered. If applicable in case of manual mixing removal from bubble formation should be indicated by a vibrator. After a curing period of approx. 30 minutes at a room temperature of 23°C the model can be replaced and should be left another few minutes to release probably existing reproductions of undercuts.

## Important working hints

Silicones are high temperature sensitive. Please consider the recommended working and storage temperature.

## Technical datas

Dosing:	1:1
Colour code:	blue
Working time:	ca. 7 minutes
Setting time:	ca. 25 minutes
Hardness:	Shore A 17
Density:	1,05 g / cm <sup>3</sup>
Viscosity:	ca. 2.400 mpA s
Linear dimension change:	< 0,2%
Tear resistance (DIN 53 504 S 1):	3,1 N/mm <sup>2</sup>
Elongation at tear (DIN 53 504 S 1):	308 %
Continuing tear resistance (ASTM D 624 B):	3,7 N/mm