Product: **REPLISIL 22 N**
Dubliersilikon / duplicating silicone

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Product: **REPLISIL 22 N**
Dubliersilikon / duplicating silicone

Instruction for use

Please read carefully and retain for future reference

Working
room temperature
at 23° C

Delivery form:
2x1 Kg bottles
2x5 Kg cans

2x6 Kg cans
2x25 Kg cans

Indications for use

REPLISIL 22 N is an addition-curing 1:1 duplicating silicone with final hardness of Shore A 22, low viscous with best mechanical characteristics and offer an extremely high accuracy in dimension and design of the duplicating form.

Working instructions

REPLISIL component A and B must be mixed at room temperature in proportion 1:1 approx. 1 min. until an uniform colour is reached. Pour it from a height of approx. 30 cm constantly onto the dry plaster model in the flask until it's all round covered. If applicable in case of manual mixing removal from bubble formation should be indicated by a vibrator. After a curing period of approx. 30 minutes at a room temperature of 23°C the model can be replaced and should be left another few minutes to release probably existing reproductions of undercuts.

Important working hints

Silicones are high temperature sensitive. Please consider the recommended working and storage temperature.

Technical datas

Dosing:	1:1
Colour code:	pink
Working time:	ca. 7 minutes
Setting time:	ca. 25 minutes
Hardness:	Shore A 22
Density:	1,05 g / cm ³
Viscosity:	ca. 2.500 mpA s
Linear dimension change:	< 0,2%
Tear resistance (DIN 53 504 S 1):	3,0 N/mm ²
Elongation at tear (DIN 53 504 S 1):	400 %
Continuing tear resistance (ASTM D 624 B):	5,0 N/mm

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Product: **REPLISIL 22 S**
Dubliersilikon / duplicating silicone

Instruction for use

Please read carefully and retain for future reference

Working
room temperature
at 23° C

Delivery form:
2x1 Kg bottles
2x5 Kg cans

2x6 Kg cans
2x25 Kg cans

Indications for use

REPLISIL 22 N is an addition-curing 1:1 duplicating silicone with final hardness of Shore A 22, low viscous with best mechanical characteristics and offer an extremely high accuracy in dimension and design of the duplicating form.

Working instructions

REPLISIL component A and B must be mixed at room temperature in proportion 1:1 approx. 1 min. until an uniform colour is reached. Pour it from a height of approx. 30 cm constantly onto the dry plaster model in the flask until it's all round covered. If applicable in case of manual mixing removal from bubble formation should be indicated by a vibrator. After a curing period of approx. 10 minutes at a room temperature of 23°C the model can be replaced and should be left another few minutes to release probably existing reproductions of undercuts.

Important working hints

Silicones are high temperature sensitive. Please consider the recommended working and storage temperature.

Technical datas

Dosing:	1:1
Colour code:	pink
Working time:	ca. 3,5 minutes
Setting time:	ca. 10 minutes
Hardness:	Shore A 22
Density:	1,05 g / cm ³
Viscosity:	ca. 2.500 mpA s
Linear dimension change:	< 0,2%
Tear resistance (DIN 53 504 S 1):	3,0 N/mm ²
Elongation at tear (DIN 53 504 S 1):	400 %
Continuing tear resistance (ASTM D 624 B):	5,0 N/mm